


## The Access Keypad

The Keypad provides four separate functions:

- Local audible and visual alarms
- Bypass function to escort residents through the protected exit
- Alarm reset when the Controller is in continuous alarm mode or resets the bypass sequence
- Controller status indication via the LED indicators

The Controller may be bypassed or reset at an exit point by entering a 3- to 7-digit code at the keypad.

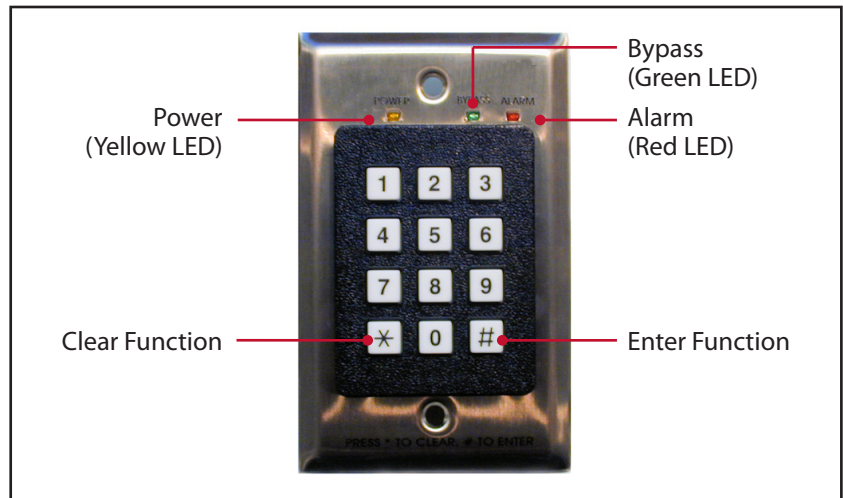

Status Indicator	Condition	Cause	Action
<b>Power</b>	OFF	Power is off	Turn Power ON
	Solid Yellow	Power ON, Ready Mode	None; system ready
<b>Bypass</b>	Flashing Green/Red + Beeping tone	Bypass Mode	Escort resident immediately
<b>Alarm</b>	Flashing Red + Beeping tone	Alarm Mode	Investigate; reset alarm

## Escorting Residents (Bypass)

1. Bring the resident to the exit, ensuring that he or she is not in the detection zone.
2. Enter the <BYPASS CODE> on the keypad.
3. Press <#> on the keypad.
4. Escort the resident through the exit.

Note: There are three automatic timers enabled by the system when the bypass function is activated:

- **Bypass Timeout:** You have 10 seconds to enter the detection zone after the bypass code is entered. If no tag enters the zone within the 10-second period, the bypass is terminated.
- **Transport:** Once the tag enters the detection zone, you have 90 seconds to transport the resident through the zone before the alarm is activated.
- **Rearm:** After the tag is through the zone or the door closes, the system automatically resets itself in 8 seconds.


### Reset an Alarm

- Enter the <Reset Code> on the keypad and press <#>.

### Clear or Correct an Entry

- To clear or correct an entry, press <\*> on the keypad.

## Optional ANN6L Six-Zone Annunciator (AGEAN01-060, North America only)

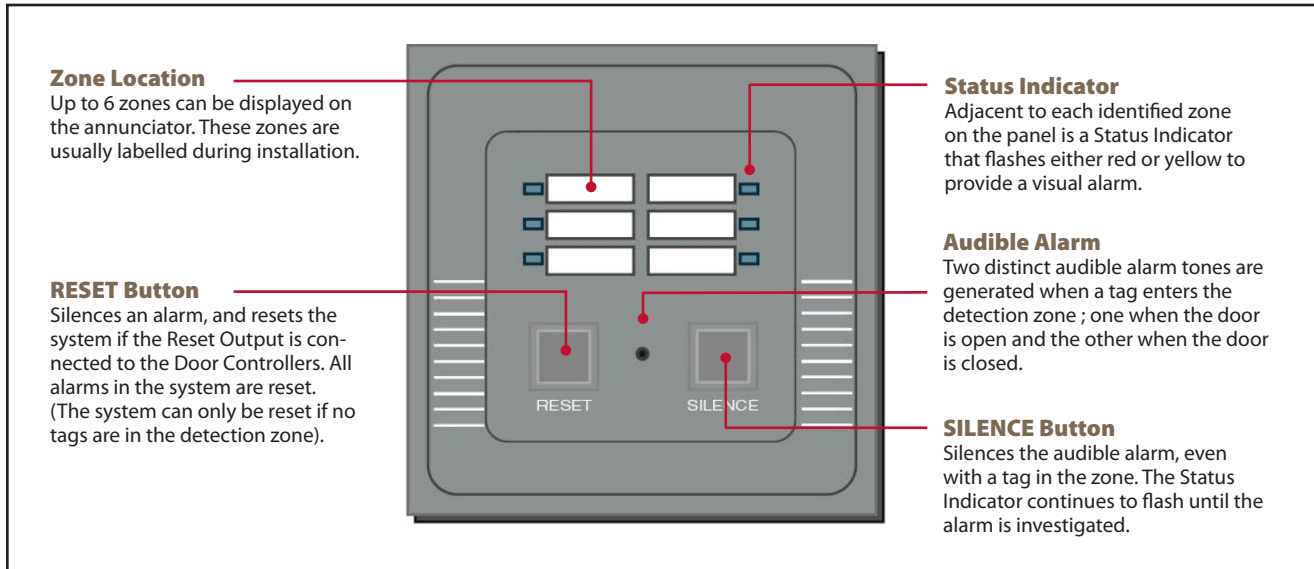

Status Indicator	Audible Tone	Door Status	Cause	Action
<b>Standby</b>	None	Closed	No tag in zone	None required
<b>Flashing YELLOW</b>	Warning Tone	Closed	Tag in zone	1. Investigate cause 2. Ensure resident safety 3. Reset panel
<b>Flashing RED</b>	Alarm Tone	Open	Tag in or passing through zone	1. Investigate cause 2. Ensure resident safety 3. Reset panel

- To silence the audible alarm, press <SILENCE>. Note: The audible alarm is silenced, but the RED indicator continues to flash.
- To reset the audible alarm, press <RESET>. Note: The RESET button only works if there are no tags in the detection zone.

### Warranty

Stanley Healthcare Solutions' RoamAlert ECO hardware is warranted against defects in materials and workmanship and shall perform in accordance with published specifications for 1 year from the date of installation.

Stanley Healthcare Solutions' warranty is limited solely to the repair or replacement of the defective part or product. Stanley Healthcare Solutions reserves the right to change product specifications without notice.

### Copyright Information

All content herein is the property of Stanley Black & Decker, Inc., its Affiliates, or their content suppliers and is protected by United States and international copyright laws. The compilation of all content is likewise the exclusive property of Stanley Black & Decker, Inc. (or the Affiliate identified in any copyright notice) and is protected by United States and international copyright laws.

The trademarks, service marks and logos (the "Trademarks") used and displayed in this publication are registered and unregistered Trademarks of Stanley Black & Decker, Inc., its Affiliates and others. Nothing herein should be construed as granting, by implication, estoppel or otherwise, any license or right to use any Trademark displayed herein without the prior consent of the Trademark owner. Stanley and the Stanley design are two of the trademarks owned by Stanley Black & Decker, Inc. and/or its Affiliates ("Stanley Black & Decker Trademarks"). Stanley Black & Decker Trademarks may not be used in connection with any product or service that is not manufactured by or under license from Stanley Black & Decker, Inc. or its appropriate Affiliate.